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Device identification:

- **CAREBAG BABY**, Potty Liner, ref 7711190

These single-use devices consist of a bag with drawstrings that contain a super-absorbent pad.
Presentation: box of 20 single bags

Indications:

This device enables excreta to be collected safely and protect the support on which they are placed. They are intended for nursing teams or caregivers who take care of babies and infant, as well as in the context of infectious isolations in hospitals.

Composition:

- Bag: Polyethylene,
- Pad: Cellulose-based support + sodium acrylate copolymers.

Performance and mechanism of action:

In contact with the SAP super-absorbent component (contained in the pad), the excreted materials gel. The SAPs are polymers that can absorb and retain large volumes of liquids relative to their own mass. They remain insoluble in the liquid that they absorb (due to their reticulated structure).

The absorption capacity of the pads contained in these CAREBAG BABY devices is 200 mL.

CAREBAG® products ensure safe collection, transport and disposal of excreta, and help prevent the risk of nosocomial infections and cross-contamination.

Instructions for use:

1. Open the box and take a bag,
2. Open the bag and turn it upside down to position the pad inside with the frog oriented lengthwise.
3. Place the bag on the support (child potty), with the pad at the bottom of the device,
4. Use the support (child potty) lined with the device,
5. Wait for contents to turn into gel,
6. Remove the bag from the device (child potty)
7. Close the bag with a knot.

Warnings, precautions for use and contraindications:

- Single use. If reused, there is a risk of insufficient or no absorption and therefore potential contamination and discomfort for the patient.
- In case the eyes and mucous membranes come in contact with the pad, rinse well with water. In the event of persistent discomfort, contact your doctor.
- Adult supervision is required for use.
- Do not detach the pad from the bag.
- Do not allow contact with damaged skin.
- Keep out of the reach of children.

Side effects:

Any serious incident in connection with the device must be reported to the manufacturer and the competent authority of the EU member state where the user and/or patient resides.

Storage and disposal:

The product must be protected from moisture and kept away from sunlight. Dispose of in household waste, or according to the waste management protocol in force in the facility.

Publication date: 01/2026

Single use only

Protect from moisture

Keep away from sunlight

See the instructions for use

Medical Device

Manufacturer
CLEANIS SASU
22 avenue Aristide Briand, 94110 Arcueil
www.cleanis.eu/en

Cleanis
CREATIVE HYGIENE

Group subsidiary **INNOTHERA**