CAREBAG

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Destination :

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24-HOUR URINE COLLECTION KIT

Purpose :

Collection of urine from a patient over 24 hours for monitoring and safe elimination

Storage conditions :

-  Keep away from sunlight
-  Fears humidity
-  Storage conditions for the powder sachets minimum 0°C
0°C
32°F
min.

CLEANIS SASU
22 avenue Aristide Briand, 94110 Arcueil (France)

Precautions for use :

- Check the expiry date before use
- Do not use in case of product failure preventing itsuse
- Do not urinate directly into the jar
- Wearing gloves is recommended
- Do not use above the maximum graduation and change bag if volume greater
- Powder sachet with water-soluble film : do not open and do not handle with wet hands
- If powder from the sachets comes into contact with eyes or mucus membranes, rinse thoroughly with water and contact a doctor if necessary
- In the powder from the sachets is swallowed, contact a doctor immediately and show them the packaging
- Dispose of the bag with the jellified urine according to the excreta management protocol in force in the establishment
- Between uses, it is advisable to disinfect the jar with a wipe
- Do not clean in the bedpan washer
- After 20 uses, dispose of the jar with recyclable plastics

Any serious incident with the medical device should be reported to CLEANIS and the competent authority of the country concerned

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
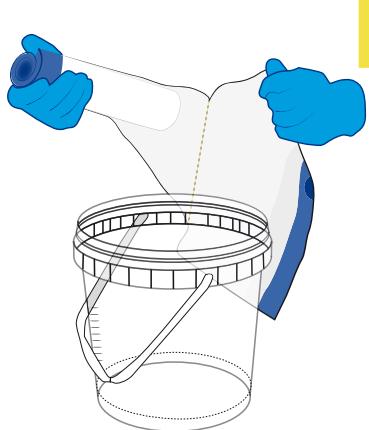
To conduct a patient's diuresis, detach a clear bag from the roll and take the bucket provided in the kit.

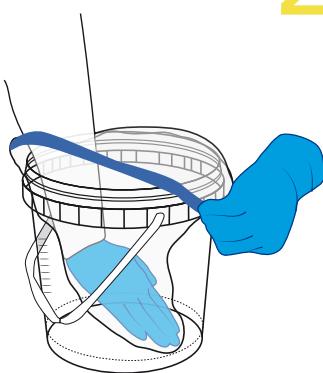Labels are available to identify the patient's diuresis.

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Place the bottom of the bag flat in the bottom of the bucket

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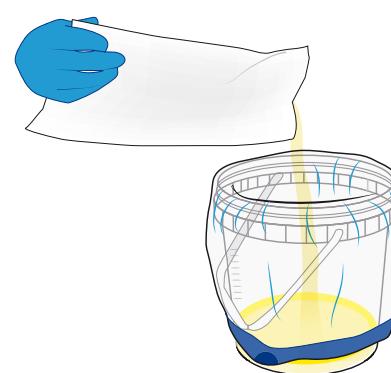
Fold the bag over the edges of the bucket.

Place one hand in the bottom of the bag and align the blue strip on the top of the bag with the black line on the bottom of the bucket.

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Pour the urine previously collected in a container into the bucket

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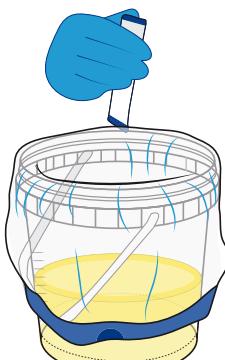
Close the bucket with the lid between uses.


Diuresis measurement : position your eyes in front of the urine level and read the correspondant graduation (with a 1.69oz uncertainty)

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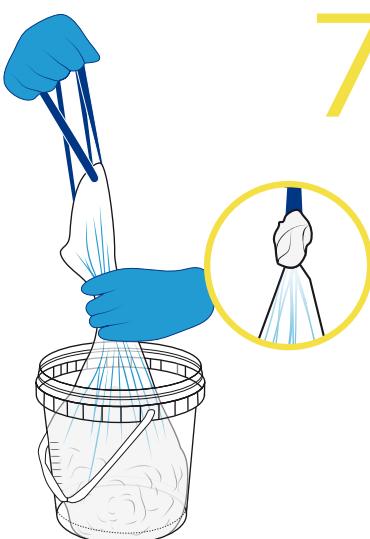Once the diuresis is complete, add the number of sachets of super-absorbent powder required to gellify the urine

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Once gelling is complete, close the bag by pulling on the ties, make an ox with the ties and remove the bag from the bucket.

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Properly dispose of the bag with gellified fluids after use, according to the protocol in force in the establishment.

Between uses, it is recommended to disinfect the bucket with a wipe. After 20 uses, dispose of the bucket with the recyclable plastics.