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These single-use devices are composed of a liner with drawstrings containing a super-absorbent pad.
Presentation : roll of 20 pre-cut devices.

Indications:

These devices enable excreta to be collected safely and protect the supports on which they are placed. They are intended for nursing teams or caregivers who take care of bedridden or disabled persons, as well as in the context of infectious isolations in hospitals.

Composition:

Liner : Polyethylene,

Pad : Cellulose based support + sodium polyacrylate.

Performances & mechanism of action:

Upon contact with the super-absorbent SAP component (contained in the pad) the excreted matter gel. SAPs are polymers that absorb and retain large volumes of liquid compared to their own mass. They remain insoluble in the liquid they absorb (thanks to their cross-linked structure).

The absorption capacity of the pads contained in these CAREBAG devices is shown in the table below :

| | CAREBAG Bedpan Liner Réf. 7711141 | CAREBAG Bedpan Liner 500 Réf. 7711148 | CAREBAG Commode Liner Réf. 7831738 | CAREBAG Toilet Bowl Liner Réf. 9538534 | CAREBAG Bedpan & Commode Pail Liner Réf. 7711160 |
|----------------------------|--|--|---|---|--|
| ABSORPTION CAPACITY | 450 mL | 500 mL | 450 mL | 450 mL | 700 mL |

CAREBAG® products ensure safe collection, transport and disposal of excreta, and help prevent the risk of nosocomial infections and cross-contamination.

Instructions for use:

1. Open the pack and take off the roll,
2. Unroll the 1st device and pull apart along perforation marks,
3. Open the liner, place it on the support (bedpan/commode pail/toilet bowl) (in case of commode pail, then fold down the seat of the chair),
4. The pad must be placed at the bottom of the device,
5. Use the bedpan/commode pail/toilet bowl liner containing the device,
6. Wait for the content to turn to gel,
7. Close the liner with a knot.

Warnings, precautions for use and contraindications:

- For single use. In case of reuse, there is a risk of insufficient or non-existent absorption, and therefore of potential contamination and discomfort for the patient,
- In case the pad comes into contact with the eye(s) or mucous membranes, rinse with plenty of tap water. In case of persistent discomfort, contact your doctor,
- Do not use on damaged skin,
- Keep away from children.

Side effects:

Any serious incident with the medical device should be notified to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Storage & Disposal :

Product must be kept dry and shall be stored away from sunlight. Dispose of in household waste, or according to the waste management protocol in place at the establishment.

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