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- **CAREBAG**, Sac vomitoire, ref 7733987

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Device identification:

- CAREBAG, Vomit bag, ref 7733987

These single-use devices consist of a bag with drawstrings that contain a super-absorbent pad.

Presentation: by 20.

Indications:

These devices make it possible to collect vomit safely. They are intended for healthcare teams or caregivers who take care of bedridden patients or persons with reduced mobility, as well as in the context of sudden episodes of nausea.

Composition:

- Bag: Polyethylene,
- Pad: Cellulose-based support + sodium acrylate copolymers.

Performance and mechanism of action:

In contact with the SAP super-absorbent component (contained in the pad), the excreted materials gel. The SAPs are polymers that can absorb and retain large volumes of liquids relative to their own mass. They remain insoluble in the liquid that they absorb (due to their reticulated structure).

The absorption capacity of the pads contained in these CAREBAG® devices is 450 mL.

CAREBAG® vomit bags secure the collection, transport and disposal of vomit and help prevent the risks of nosocomial infections and cross-contamination.

Instructions for use:

1. Open the box and take a bag,
2. Unfold the bag, and open it,
3. Vomit in the bag,
4. Wait until the contents are transformed into gel,
5. To close, pull the cords on each side of the bag, i.e. blue on one side, white on the other; tie.

Warnings, precautions for use and contraindications:

- Single use. If reused, there is a risk of insufficient or no absorption and therefore potential contamination and discomfort for the patient.
- In case the eyes and mucous membranes come in contact with the pad, rinse well with water. In the event of persistent discomfort, contact your doctor.
- Do not allow contact with damaged skin.
- Keep out of the reach of children.

Side effects:

Any serious incident in connection with the device must be reported to the manufacturer and the competent authority of the EU member state where the user and/or patient resides.

Storage and disposal:

The product must be protected from moisture and kept away from sunlight. Dispose of in household waste, or according to the waste management protocol in force in the facility.

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