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Dispositif Médical

Device identification:

- **CAREBAG**, Recycled Male urinal with holder, pack of 12: ref. 9610150
- **CAREBAG**, Male urinal, pack of 20: ref. 7733993

These single-use devices consist of a bag with drawstrings that contain a super-absorbent pad.
Presentation: by 20 or 12. The presentation with 12 incorporates cardboard holders.

Indications:

These male urinals allow urine to be collected safely. They are intended for healthcare teams or caregivers who take care of bedridden patients or persons with reduced mobility.

Composition:

- Bag: Polyethylene,
- Pad: Cellulose-based support + sodium acrylate copolymers,
- Holder: Cardboard (only for ref. 9610150)

Performance and mechanism of action:

In contact with the SAP super-absorbent component (contained in the pad), the urine gels. The SAPs are polymers that can absorb and retain large volumes of liquids relative to their own mass. They remain insoluble in the liquid that they absorb (due to their reticulated structure).

	CAREBAG Male Urinal Réf. 7733993	CAREBAG Recycled Male Urinal with holder Réf. 9610150
ABSORPTION CAPACITY	450 mL	700 mL

CAREBAG® Male Urinals secure the collection, transport and disposal of urine and help prevent the risks of nosocomial infections and cross-contamination.

Instructions for use:

1. Open the box and take a bag,
2. Unfold the bag, and open it. If the holder is included, put the holder around the bag and hold it with your hands,
3. Urinate into the bag,
4. Wait until the contents are transformed into gel,
5. To close, pull the cords on each side of the bag, i.e. blue on one side, white on the other: tie.

Warnings, precautions for use and contraindications:

- Single use. If reused, there is a risk of insufficient or no absorption and therefore potential contamination and discomfort for the patient.
- In case the eyes and mucous membranes come in contact with the pad, rinse well with water. In the event of persistent discomfort, contact your doctor.
- Do not allow contact with damaged skin
- Keep out of the reach of children.

Side effects:

Any serious incident in connection with the device must be reported to the manufacturer and the competent authority of the EU member state where the user and/or patient resides.

Storage and disposal:

The product must be protected from moisture and kept away from sunlight. Dispose of in household waste, or according to the waste management protocol in force in the facility.

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