## OPE 150

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Dispositif Médical

## OPE 150

### ABSORBENT MAT FOR TRANSPORT AND INTERVENTION

Reference : 9522214

#### Indications :

Gelmax OPE 150 is an absorbent mat intended to allow the transfer of patients and to absorb fluids (blood, *excreta*, etc.) during medical examinations and/or surgical interventions for the hygiene of the operating room. This medical device contributes to the comfort of the users and the patient and is mainly used by health professionals (emergency physicians, stretcher bearers, caregivers, etc.) or firefighters.

#### DM composition :

Gelmax OPE 150 is a multi-layer mat. Composition: Polyethylene, polypropylene, cellulose and sodium polyacrylate.

#### Mechanism of action :

Sodium polyacrylate allow the absorption of biological fluids. **The absorption capacity is equivalent to 1.8 liters of 0.9% saline solution (NaCl).**

#### Instruction for use :

Position the absorbent mat on the user support. **The super-absorbent checkered part must face upwards.**

#### Warnings, Precautions of use and Contraindications :

- Do not use for a patient over 150 kg.
- Ensure that the patient is in a lying position.
- Do not use near high voltage devices.
- To avoid tearing of the material during transfer, avoid any contact with sharp objects and avoid applying tension with fingernails.
- Do not cut the product.
- Do not reuse (*risk of insufficient absorption or even none and therefore a potential patient contamination and discomfort*).

#### Sides effects :

No side effects known.

Any serious incident with the medical device should be notified to CLEANIS and the competent authority of the concerned country.

#### Storage, disposal :

Product fear humidity and shall be stored away from sunlight. Dispose of according to the waste management protocol in the establishment.

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