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**Device identification:**

Individual water-soluble super-absorbent pouch 600ml  
Presentation: by 100

**Indications:**

Gels body fluids contained in small containers such as ostomy pouches after use and removal, suction jars...  
This device is particularly intended for healthcare professionals.

**Composition:**

Water-soluble pouch: polyvinyl alcohol (PVOH), dimensions 105x25mm.  
Powder: sodium polyacrylate

**Performance and mechanism of action:**

Upon contact with the super-absorbent SAP component (contained in the pad) the excreted matter gel. SAPs are polymers that absorb and retain large volumes of liquid compared to their own mass. They remain insoluble in the liquid they absorb (thanks to their cross-linked structure). The absorption capacity of each individual water-soluble pouch is 600ml of 0.9% saline solution.

**Instructions for use:**

1. Open the pack and take a water-soluble pouch,
2. Place the water-soluble pouch in the liquid to be gelled or in the container,
3. Allow water-soluble pouch to dissolve on contact with liquids,
4. Wait for the liquids to turn to gel,
5. Dispose of in appropriate place.

**Warnings, precautions for use and contraindications:**

- For single use. In case of reuse, there is a risk of insufficient or non-existent absorption, and therefore of potential contamination and discomfort for the patient,
- Wearing gloves is recommended.
- Do not open the pouch and do not touch with wet hands.
- Do not use this product to gel the following chemical products: strong acids (concentrations > 3%), strong bases (concentrations > 10%) and strong oxidizing agents,
- Keep away from children,
- In case the powder comes into contact with the eye(s) or mucous membranes, rinse with plenty of tap water. In case of persistent discomfort, contact your doctor,
- In case of ingestion, contact a doctor immediately.

**Side effects:**

Any serious incident with the medical device should be notified to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Storage and disposal:**

Product must be kept dry and shall be stored away from sunlight. Product must be stored at a minimum temperature of 0°C. Dispose of in household waste, or according to the waste management protocol in place at the establishment.

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